## **X - COVER - INSTRUCTION MANUAL**

X Covers are cosmetic prosthetic covers designed for limb prostheses. It is an accessory acting as a cosmetic layer for limb prosthesis. XS and XF Covers are made for people after lower limb amputation above the knee (AK) and XT Covers are for people with below-knee amputation (BK). Before putting on the cover please check the recommendations for use provided by the prosthesis manufacturer. X Cover can be put on the prosthesis by the prosthesis dealer or a prosthetic technician who is obliged to inform the user about the correct handling of the product. The X Cover can be put on by the user of the prosthesis, provided that the instruction manual is followed.

### **XS, XF and XT Covers**

If a stabilising buckle was supplied with the X Cover, put it on the prosthesis adapter and adjust the place of attachment by putting on the cover. To put it on, gently open the cover and place it on the prosthesis. Align the fastening point of the buckle in the cover's pocket and fix the buckle with the screws. Put on the X Cover and apply pressure on the cover **starting from the bottom** along the edges to connect them.

If a stabilising buckle was not supplied with the X Cover, put on the prosthesis cover. If the XS or XT Cover moves while walking, stick the supplied Velcro between the prosthesis and the place, where the cover is adjacent to the prosthesis (e.g. on the side of the knee of the pipe adapter). For XT Covers, the supplied Velcro can be attached between the socket and the cover. The place where the Velcro is attached should be thoroughly cleaned beforehand. The X Cover should be degreased only on the inside (!) with acetone, whereas the sticking points on the prosthesis should be degreased with, e.g. denatured alcohol or extraction gasoline. Please pay special attention not to use the above-mentioned chemical products on the outer surface of the cover, as the surface of the finish may be damaged.

### **GUIDELINES FOR USE**

When using the X Cover, please follow the prosthesis manufacturer's guidelines for use.

If X Cover is to be used by children, they must be instructed on how to use it safely. The cover must be maintained under the supervision of an adult.

Please be careful when kneeling (only if the design of the prosthesis allows you to do so) and if possible kneel on a soft surface (e.g. by placing something soft under the knee).

Keep away from the fire source.

Avoid subjecting the cover to heavy impacts, sharp objects and squeezing its components.

Do not expose to strong sunlight for long periods of time, do not place near a heat source.

Do not use chemicals on the outer surface of the cover.

If the prosthesis is not waterproof or made for water sports, be careful when using the X Cover when carrying out activities with water so as not to get the knee or foot prosthesis wet.

### **Maintenance**

Clean with a wet cloth.

After removing the X Cover from the prosthesis, you can clean it gently with a damp cloth or dip it in water. To remove dirt, you can use a mild liquid detergent or soap and wipe dry.

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We declare, that the device that is covered by this declaration is in conformity with:

**Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices, amending Directive 2001/83/EC, Regulation (EC) No 178/2002 and Regulation (EC) No 1223/2009 and repealing Council Directives 90/385/EEC and 93/42/EEC.**

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EU Declaration of conformity is issued under the sole responsibility of the manufacturer.

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www.nextt.pl, www.nexttprosthetics.com, e-mail: nextt@nextt.pl

ver. I X-Cover 2025.03