



RECOMMENDATIONS AND NOTES ON ORDERS

- For feet as shown below, the height measurement must include the lowest point. In this case, the cover must be cut to fit the shape of the foot.

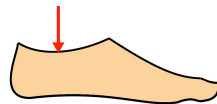

- If the cover is to have any special holes, e.g. for adjustment or for the PIN button, you should choose covers only in the MAX version.
- There should be a gap of approximately 7 mm between the Neo cover (with magnets) and the foot. For very soft feet, such as those made of fiberglass, this distance should be at least 13 mm. The cover should not touch the foot while walking.

- If the circumference of the prosthetic socket at its widest point is similar to the calf circumference (or greater), the Cover height should be as low as possible. It is important to remember that placing the Cover on the socket increases its circumference and may not look good together with the healthy leg.

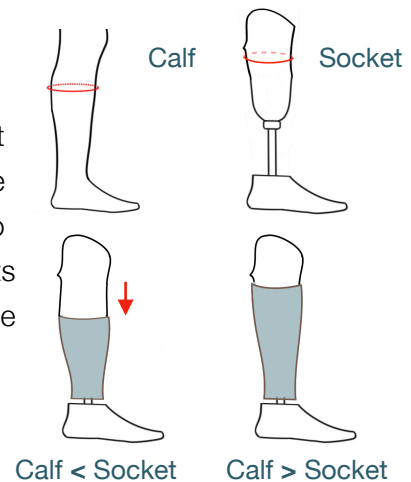

- All of our Neo covers (with magnets) can only be shortened from the bottom, using a knife or scissors. Any unevenness can be sanded down with a socket grinder.



- Before scanning smooth or shiny surfaces, such as stainless steel, it's a good idea to spray the surface with a special powder spray, such as that available from IP Division GmbH (www.ipdent.com).



PAINLESS TATTOOS



A tattoo is available for an additional fee. You can choose a pre-made tattoo from our website or design your own.

For your own tattoo, please send us a graphic file. However, there are some rules:

- The photo or drawing must be on a white background.
- Vertical orientation means the height of the drawing should be at least twice its width.
- The resolution must be sufficient to avoid graininess—files over 300 KB are generally suitable.



SCANNER

To precisely create covers, we need accurate data. Often, a scan of the prosthesis and calf is also required. We recommend an inexpensive scanner, a built-in device for iPads or iPhones, from **Structure**.

